



# Comfiwave<sup>®</sup>

Compression to drift off in



## What is *Comfiwave*<sup>®</sup>?

Comfiwave is a patented **compression garment**, designed to complement the treatment a patient receives during waking hours.

Comfiwave gives a therapeutic pressure of **15-21mmHg**.

**Easy donning and doffing** gives Comfiwave an innovative versatility, providing both night-time compression and a comfortable solution when resting in the daytime.



**Below Knee**  
Page 10 ▶



**Thigh High**  
Page 12 ▶



**Toecap**  
Page 14 ▶



**Sleeve**  
Page 16 ▶



**Sleeve + Mitten**  
Page 18 ▶



**Glove**  
Page 20 ▶



**Glove to Elbow**  
Page 22 ▶



**Glove to Axilla**  
Page 24 ▶



**Abdominal**  
Page 26 ▶



**Breast Band**  
Page 28 ▶



**Neck**  
Page 30 ▶



## For night time

Oedema related swelling doesn't stop overnight but traditional compression garments are not best suited to being worn in bed. Continue compression therapy overnight in supreme comfort with Comfiwave's soft and breathable fabric providing gentle compression while you sleep.



## For any time

Comfiwave garments use proprietary 'low-resistance' yarns which allow them to be easily donned and doffed. Perfect for relaxing during the day.

The breathability of the fabric means that Comfiwave is suitable for use all year round, being cooler to wear than conventional hosiery.



# Whole-garment knitting technology

Easy colour coded sizing system

e.g.  = Large, Long

Comfiwave's wave patterning mimics the pressure differential massage effect that is ordinarily found in cut-foam, chip-bag style garments.

Comfiwave lower limb garments utilise a stiffened and shaped ankle to prevent 'cutting in' of the anterior ankle

The lateral toe section has also been knitted to stretch more, which can prevent pressure sores on the outside of the toes

Heel section helps prevent slippage



# Comfiwave® features



## Easy to don and doff

Patented, high-elastic knit makes Comfiwave easy to put on and take off



## Made from a soft, undyed cotton

Soft cotton against the skin and seamless design ensures maximum comfort



## Breathable fabric

Comfiwave's wave patterning gives it high air permeability for coolness and stretch

# *Comfiwave*<sup>®</sup> indications



Provides safe, night-time compression

Complements treatment a patient receives during waking hours



Can be used in palliative care



Can be worn under easywrap or bandaging



**I love this garment.** I have been wearing it overnight for the past week and it has made a massive difference!

- Lymphoedema patient

**Really comfortable to wear at night.** Unlike other products doesn't make my leg hot when temperature is high. Easy to put on and remove. Excellent product.

- Lymphoedema patient



“ Patients reported that Comfiwave was comfortable and easy to use, which itself would improve concordance with treatment and self-management of oedema. ”

“ ...this type of product can be used for stand-alone treatment or as an adjunct to other treatment modalities and for 24 hours a day if needed. ”

Download  
clinical case  
studies and article  
[www.comfiwave.info](http://www.comfiwave.info)

British Journal of  
Community Nursing

**PRODUCT FOCUS**

**Table 1. Mrs S's limb volume and circumference measurements at baseline and 2 weeks after using Comfiwave**

Circumference (cm)	A	C	D	F	G	Limb volume A	Unaffected	% difference
Baseline	17.5	17.5	25	29	31.5	2329	1954	+16%
After 2 weeks	17	15.8	24	27.5	31	2138	1953	-8%

through her teenage years with bilateral lower leg oedema and recurrent episodes of cellulitis.

On initial assessment, there was evidence of bilateral lymphoedema below the knees, with both feet showing positive Stemmer's sign, mild hyperkeratosis and post-cellulitis skin discoloration on the left gaiter area. The left leg was 30% bigger than the right, with an ankle circumference of 51 cm and calf of 65 cm. There were distinct skin folds distorting the shape around her left ankle, as well as foot oedema, which was restricting footwear (Figure 4). The limb size also restricted clothing, and the patient was only able to wear very loose trousers or long skirts.

Initial advice of skin care, exercise, elevation and weight management were discussed, and 2 weeks of daily treatment were booked. Miss H was signed off work by her GP to commence in October 2018 and involved alternating days of MLD or intermittent compression using Haddesham 1 to 2 times a day, followed by limb volume measurements.

through her teenage years with bilateral lower leg oedema and recurrent episodes of cellulitis.

On initial assessment, there was evidence of bilateral lymphoedema below the knees, with both feet showing positive Stemmer's sign, mild hyperkeratosis and post-cellulitis skin discoloration on the left gaiter area. The left leg was 30% bigger than the right, with an ankle circumference of 51 cm and calf of 65 cm. There were distinct skin folds distorting the shape around her left ankle, as well as foot oedema, which was restricting footwear (Figure 4). The limb size also restricted clothing, and the patient was only able to wear very loose trousers or long skirts.

Initial advice of skin care, exercise, elevation and weight management were discussed, and 2 weeks of daily treatment were booked. Miss H was signed off work by her GP to commence in October 2018 and involved alternating days of MLD or intermittent compression using Haddesham 1 to 2 times a day, followed by limb volume measurements.

**Figure 5. Miss H wearing Comfiwave on the left leg**

**Figure 6. Miss H's legs after using Comfiwave for 2 weeks**

**PRODUCT FOCUS**

within 'wave' patients' to the y were improve sure of hat this pent or as 24 hours efficacy in decreasing edema and lent, family

and a prospective 3-541, https://doi. id deep with using Comm Nurs 2017;


Author J Examination is. Nurse registration 429-435. https://doi. 8 | JoVidPub PPI Publications



## Comfiwave® Below Knee

	SHORT		REGULAR		LONG
<b>S</b>	CW1-AD2S 422-1412	■	CW1-AD2R 422-1446	■	CW1-AD2L 422-1438
<b>M</b>	CW1-AD3S 422-1362	■	CW1-AD3R 422-1420	■	CW1-AD3L 422-1479
<b>L</b>	CW1-AD4S 422-1370	■	CW1-AD4R 422-1396	■	CW1-AD4L 422-1453
<b>XL</b>	CW1-AD5S 422-1388	■	CW1-AD5R 422-1404	■	CW1-AD5L 422-1461

Drug Tariff codes shown in red

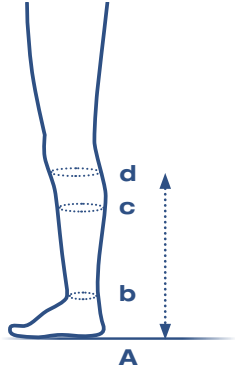

Circumferences cm

**S**

**M**

**L**

**XL**

**c** WIDEST CALF

25 - 32

30 - 38

36 - 46

44 - 56

**b** ANKLE

17 - 22

20 - 26

22 - 30

27 - 35

Lengths cm

**SHORT**

**REGULAR**

**LONG**

**A-d** FLOOR TO BELOW KNEE

32 - 37

37 - 42

42 - 48



**b**  
Ankle



**c**  
Widest Calf




**A-d**  
Floor to below knee



## Comfiwave® Thigh High

	SHORT	REGULAR	LONG
<b>S</b>	CW1-AG2S 422-1487	CW1-AG2R 422-1529	CW1-AG2L 422-1586
<b>M</b>	CW1-AG3S 422-1560	CW1-AG3R 422-1537	CW1-AG3L 422-1495
<b>L</b>	CW1-AG4S 422-1552	CW1-AG4R 422-1511	CW1-AG4L 422-1545
<b>XL</b>	CW1-AG5S 422-1503	CW1-AG5R 422-1578	CW1-AG5L 422-1594
<b>XXL</b>	CW1-AG6S	CW1-AG6R	CW1-AG6L

Drug Tariff codes shown in red

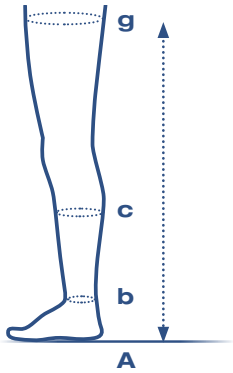

Circumferences cm

S

M

L

XL

XXL

**g** THIGH TOP

40 - 48

46 - 56

54 - 65

62 - 75

72 - 92

**c** WIDEST CALF

25 - 32

30 - 38

36 - 46

44 - 56

52 - 64

**b** ANKLE

17 - 22

20 - 26

22 - 30

27 - 35

27 - 35

Lengths cm

SHORT

REGULAR

LONG

**A-g** FLOOR TO THIGH TOP

64 - 70

70 - 77

77 - 84



**b**  
Ankle



**c**  
Widest Calf



**g**  
Thigh Top



**A-g**  
Floor to thigh top





## Comfiwave® Toecap

REGULAR

M

CW1-ZA3



L

CW1-ZA4





Circumferences cm

M

L

**a** BALL OF FOOT

20 - 27

26 - 34



**a**  
Ball of foot



## Comfiwave® Sleeve

### SHORT

### REGULAR

S

CW1-CG2S  
422-1198



CW1-CG2R  
422-1172



M

CW1-CG3S  
422-1180



CW1-CG3R  
422-1149



L


CW1-CG4S  
422-1164



CW1-CG4R  
422-1156



Drug Tariff codes shown in red

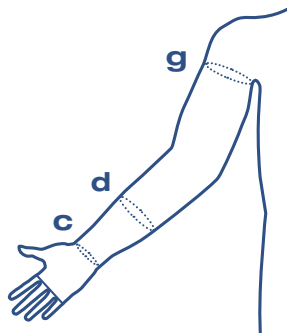

Circumferences cm		S	M	L
<b>g</b>	<b>AXILLA</b>	25 - 30	30 - 40	40 - 50
<b>d</b>	<b>MID-POINT OF FOREARM</b>	23 - 26	26 - 30	30 - 38
<b>c</b>	<b>WRIST</b>	13 - 18	18 - 22	22 - 28

Lengths cm		SHORT	REGULAR
<b>c-g</b>	<b>WRIST TO AXILLA</b>	36 - 42	40 - 46



**c**  
Wrist



**d**  
Mid-point of  
forearm



**g**  
Axilla



**c-g**  
Wrist to axilla




## Comfiwave® Sleeve + Mitten

	SHORT		REGULAR	
<b>S</b>	CW1-AH2S 422-1214	■	CW1-AH2R 422-1230	■
<b>M</b>	CW1-AH3S 422-1206	■	CW1-AH3R 422-1248	■
<b>L</b>	CW1-AH4S 422-1222	■	CW1-AH4R 422-1255	■

Drug Tariff codes shown in red



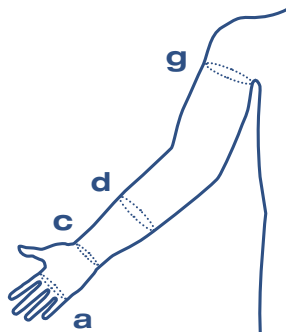

Circumferences cm		S	M	L
<b>g</b>	AXILLA	25 - 30	30 - 40	40 - 50
<b>d</b>	MID-POINT OF FOREARM	23 - 26	26 - 30	30 - 38
<b>c</b>	WRIST	13 - 18	18 - 22	22 - 28
<b>a</b>	PALM AT BASE OF FINGERS	16 - 22	18 - 24	20 - 26

Lengths cm		SHORT	REGULAR
<b>c-g</b>	WRIST TO AXILLA	36 - 42	40 - 46



**a**  
Palm at base of fingers



**c**  
Wrist



**d**  
Mid-point of forearm



**g**  
Axilla



**c-g**  
Palm at base of fingers to axilla



# Comfiwave® Glove

REGULAR

S

CW1-ZC2



M


CW1-ZC3



L

CW1-ZC4



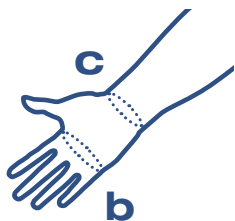

Circumferences cm

**S**

**M**

**L**

**b** BASE OF THUMB

17 - 22

20 - 25

23 - 28

**c** WRIST

14 - 20

16 - 24

22 - 28



**b**  
Base of thumb  
webbing



**c**  
Wrist




# Comfiwave®

## Glove to Elbow

REGULAR

S	CW1-ZE2	■
M	CW1-ZE3	■
L	CW1-ZE4	■

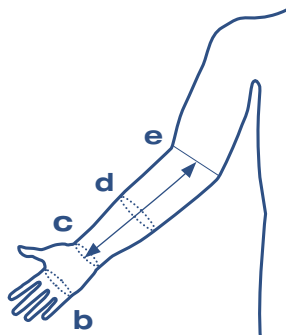

Circumferences cm

**S**

**M**

**L**

<b>b</b>	<b>BASE OF THUMB WEBBING</b>	17 - 22	20 - 25	23 - 28
<b>c</b>	<b>WRIST</b>	14 - 20	16 - 24	22 - 28
<b>d</b>	<b>MID - POINT OF FOREARM</b>	23 - 28	26 - 32	30 - 38

Lengths cm

**REGULAR**

<b>c-e</b>	<b>WRIST TO ELBOW</b>	20 - 24
------------	-----------------------	---------



**b**  
Base of thumb webbing



**c**  
Wrist



**d**  
Mid-point of forearm



**c-e**  
Wrist to elbow  
GLOVE TO ELBOW  
AND AXILLA ONLY





# Comfiwave®

## Glove to Axilla

SHORT

REGULAR

S

CW1-ZH2S

CW1-ZH2R

M


CW1-ZH3S

CW1-ZH3R

L

CW1-ZH4S

CW1-ZH4R

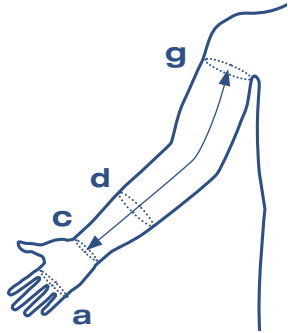

Circumferences cm		<b>S</b>	<b>M</b>	<b>L</b>
<b>g</b>	AXILLA	25 - 30	30 - 40	40 - 50
<b>d</b>	MID-POINT OF FOREARM	23 - 26	26 - 30	30 - 38
<b>c</b>	WRIST	13 - 18	18 - 22	22 - 28
<b>a</b>	PALM AT BASE OF FINGERS	16 - 22	18 - 24	20 - 26

Lengths cm		<b>SHORT</b>	<b>REGULAR</b>
<b>c-g</b>	WRIST TO AXILLA	36 - 42	40 - 46



**b**  
Base of thumb  
webbing



**c**  
Wrist



**d**  
Mid-point of  
forearm



**c-e**  
Wrist to elbow  
GLOVE TO ELBOW  
AND AXILLA ONLY




## Comfiwave® Abdominal Band

M	CW1-HT3	■
L	CW1-HT4	■
XL	CW1-HT5	■



## Comfiwave® Thigh + Hip

	SHORT		REGULAR	
S	CW1-ET2S	■	CW1-ET2R	■
M	CW1-ET3S	■	CW1-ET3R	■
L	CW1-ET4S	■	CW1-ET4R	■
XL	CW1-ET5S	■	CW1-ET5R	■

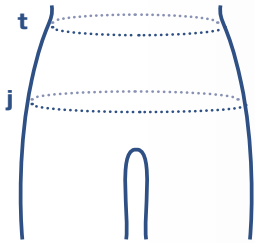

Circumferences cm

**t** WAIST

**M**

60 - 75

**L**

70 - 85

**XL**


90 - 120

**j** HIPS

80 - 100

90 - 105

110 - 140

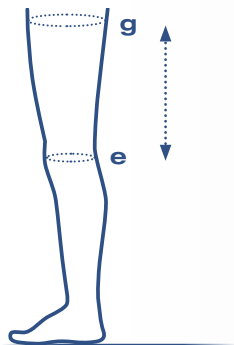

Circumferences cm

**g** THIGH TOP

**S**

40 - 48

**M**

46 - 56

**L**

54 - 65

**XL**

62 - 75

**e** KNEE

27 - 35

33 - 39

41 - 50

48 - 60


**SHORT**

**REGULAR**

**e-g** KNEE TO THIGH TOP

20 - 26

25 - 31

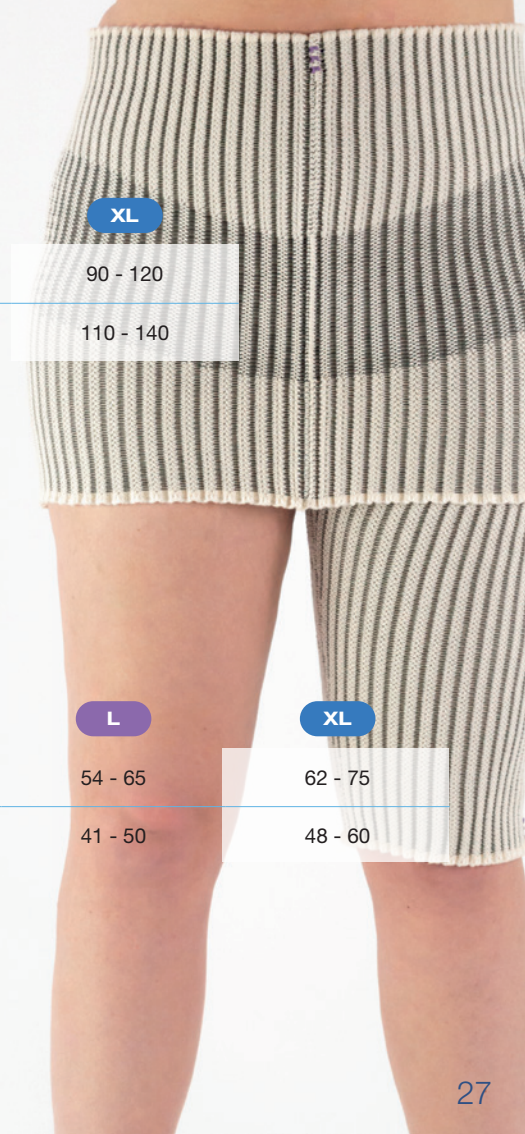






## Comfiwave® Breast Band

S	CW1-TN2	••••
M	CW1-TN3	••••
L	CW1-TN4	••••
XL	CW1-TN5	••••



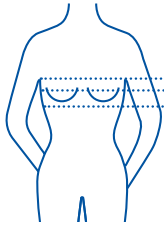

Circumferences cm	S	M	L	XL
ABOVE BREAST	70 - 88	80 - 102	90 - 116	100 - 130
OVER BREAST	70 - 92	80 - 108	90 - 126	100 - 144
BELOW BREAST	70 - 80	80 - 92	90 - 104	100 - 116



Above Breast



Over Breast



Under Breast



## Comfiwave® Neck

SHORT

REGULAR

S


CW1-HN2S

CW1-HN2R

M

CW1-HN3S

CW1-HN3R

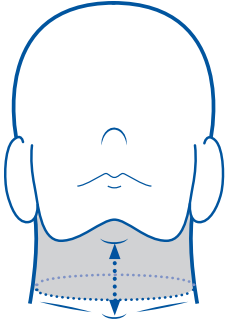

**BASE OF NECK**

**S**

32 - 38

**M**

37 - 43

**SHORT**

5-7

**REGULAR**

7-9

**BASE OF NECK  
TO TOP OF NECK**



Base of Neck



Base of Neck  
to Top of Neck

*Comfiwave*<sup>®</sup>

Haddenham Healthcare Ltd  
4D Drakes Drive  
Long Crendon, HP18 9BA  
United Kingdom



[www.comfiwave.info](http://www.comfiwave.info)



01844 208842



[sales@hadhealth.com](mailto:sales@hadhealth.com)